


AMNIOTIC MEMBRANE ALLOGRAFT TECHNIQUE GUIDE



Preparation & Application



1. Open product package and remove peel-pouch containing the graft.

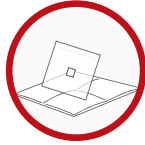


2. Using aseptic technique, peel open outer pouch and place inner pouch onto sterile field.



3. When ready to use, peel open pouch to expose the graft.



4. Remove graft using dry, sterile gloves or forceps.

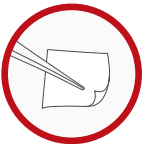


5. Use forceps to apply the graft onto the site. Achieve full contact. Graft adheres naturally without fixation.



6. Graft may be cut with scissors to apply over multiple sites.



7. If desired, graft may be moistened prior to application with sterile saline, for tight or hard to reach areas.

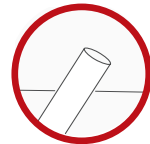


8. For minimally invasive applications, graft may be hydrated in saline, and placed down the port. Use closed graspers to nudge graft into place.


Topical Wounds



1. Prepare wound site by removing any non-viable tissue and excess fluid.

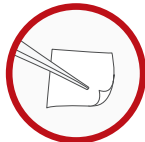

2. Apply graft onto the wound, achieving full contact.

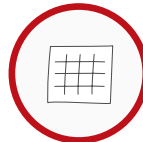

3. Apply non-adherent dressing, with multi-layer compression bandage. Keep site moist – not dry, not wet.



4. Product may be re-applied as needed. Re-application at 14-21 days, if required, is typical.

Sizing




Select the appropriate graft size to completely cover the wound site.

Post-Treatment



After use, affix Tissue ID Label from box to the included Tracking Card, and return to Skye customer service.

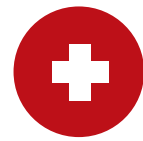

Follow standard post-treatment care protocol. Dress topical wounds with non-adherent dressing and multi-layer compression bandage.



Physicians may prescribe anti-inflammatories or other medications as they see fit.